

INSTRUCTIONS FOR USE

0401 Cervical Collar (Low Density)

CAUTION:

- 1)User should inspect their neck along with the product prior and after each use.
- 2)Only a prescribing Physician could apply for any modifications of the product.
- 3)The patient is free to adjust the product for comfort or functional purposes.
- 4)The neck must be in neutral position and not hyperextended.
- 5)In cases of pain,skin irritation or swelling the patient must stop the use of the collar.

FITTING INSTRUCTIONS:

- 1)Unstick the hook and loop.
- 2)Place the collar around the neck with the V-shape surface below the chin.
- 3)Having customized the product to a comfortable position, stick again the hook and loop.

WASHING INSTRUCTIONS:

The collar may be cleaned in hand wash at 30 degrees.

Air dry only,ironing is forbidden.

Do not wash it in washing machine or professional cleaning.

Avoid bleaching or strong detergents.

INDICATIONS:

- 1)Cervical pain
- 2)Cervical Syndrome

COMPOSITION:FOAM-COTTON

AVAILABLE COLOURS:WHITE

AVAILABLE SIZES:S-M-L-XL-XXL

ΟΔΗΓΙΕΣ ΧΡΗΣΗΣ

0401 Αυχενικό κολάρο (μαλακό)

ΠΡΟΣΟΧΗ:

- 1) Ο χρήστης θα πρέπει να ελέγχει τον αυχένα του και το προϊόν πριν και μετά από κάθε χρήση.
- 2) Μόνο ένας θεράπων ιατρός θα μπορούσε να εφαρμόσει τυχόν τροποποιήσεις στο προϊόν.
- 3) Ο ασθενής είναι ελεύθερος να προσαρμόσει το προϊόν για την άνεση ή λειτουργικούς σκοπούς.
- 4) Ο αυχένας πρέπει να είναι σε ουδέτερη θέση και όχι σε υπερέκταση.
- 5) Σε περιπτώσεις πόνου, ερεθισμού του δέρματος ή πρήξιμο του ασθενούς πρέπει να σταματήσει τη χρήση του κολάρου.

ΟΔΗΓΙΕΣ ΤΟΠΟΘΕΤΗΣΗΣ:

- 1) Ξεκollήστε το velcro.

- 2) Τοποθετείστε το κολλάρο γύρω από τον αυχένα με την επιφάνεια σχήματος V κάτω από το πηγούνι.
- 3) Έχοντας προσαρμόσει το προϊόν σε μια άνετη θέση, κολλήστε και πάλι το velcro.

ΟΔΗΓΙΕΣ ΠΛΥΣΙΜΑΤΟΣ:

Το κολλάρο μπορεί να καθαριστεί σε πλύσιμο στο χέρι στους 30 βαθμούς.

Ξηρό αέρα μόνο, απαγορεύεται το σιδέρωμα.

Να μην πλένετε στο πλυντήριο ή με επαγγελματικό καθαρισμό.

Αποφύγετε την λεύκανση ή ισχυρά απορρυπαντικά.

ΕΝΔΕΙΞΕΙΣ:

1)Αυχεναλγίες

2)Αυχενικό Σύνδρομο

ΣΥΝΘΕΣΗ:FOAM-COTTON

ΔΙΑΘΕΣΙΜΑ ΧΡΩΜΑΤΑ:ΛΕΥΚΟ

ΔΙΑΘΕΣΙΜΑ ΜΕΓΕΘΗ:S-M-L-XL-XXL